


QuantiFERON-TB Gold Plus

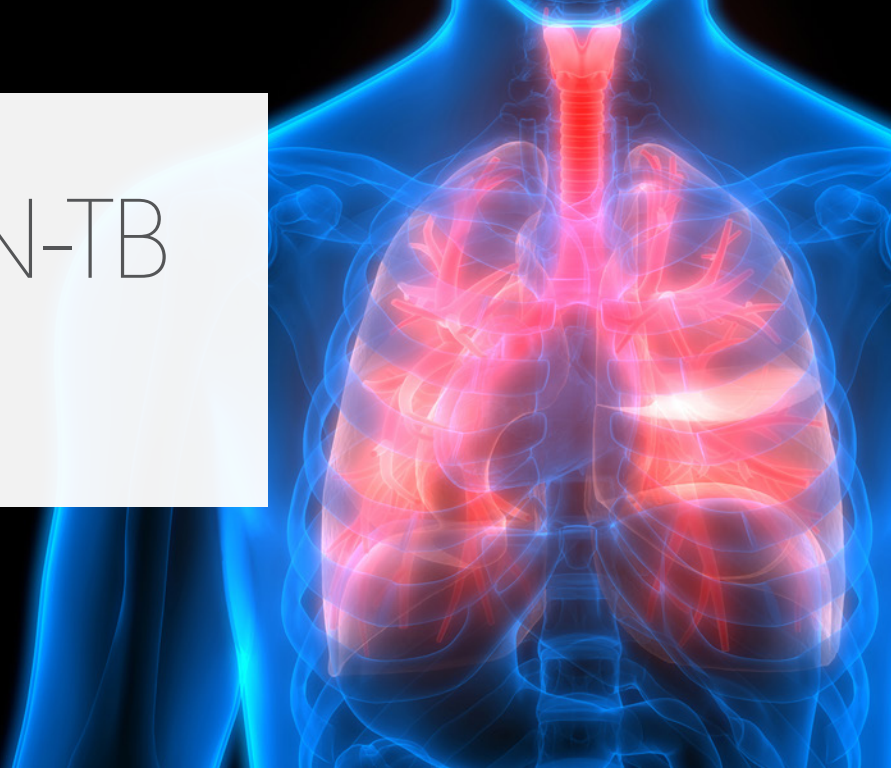

ARUP's QuantiFERON-TB Gold Plus assays offer optimized analyses with tuberculosis (TB)-specific antigens that elicit both CD8+ and CD4+ T-cell responses—enabling a more comprehensive assessment of cell-mediated immune response to TB infection.

Features and Benefits

- Convenient one-tube option—specimen preparation and processing is performed entirely at ARUP vs. the client site
- Lower percentage of indeterminate results means less risk of redraw
- CDC recommends interferon- γ release assay (IGRA) over tuberculin skin test (TST)
- Electronic orders and interfaced results ensure lower rates of transcription error

For more information

- QuantiFERON: aruplab.com/testing/quantiferon
- QuantiFERON-TB Gold Plus Special collection and preparation instructions: aruplab.com/testing/quantiferon#collection
- Email inquiries to: ProductManagersTechnical@aruplab.com


ARUP Laboratories
offers QuantiFERON-TB
Gold Plus blood
collection with a single
lithium-heparin tube.

ANSWERS
START HERE



QuantiFERON-TB Gold Plus

Collection Instructions

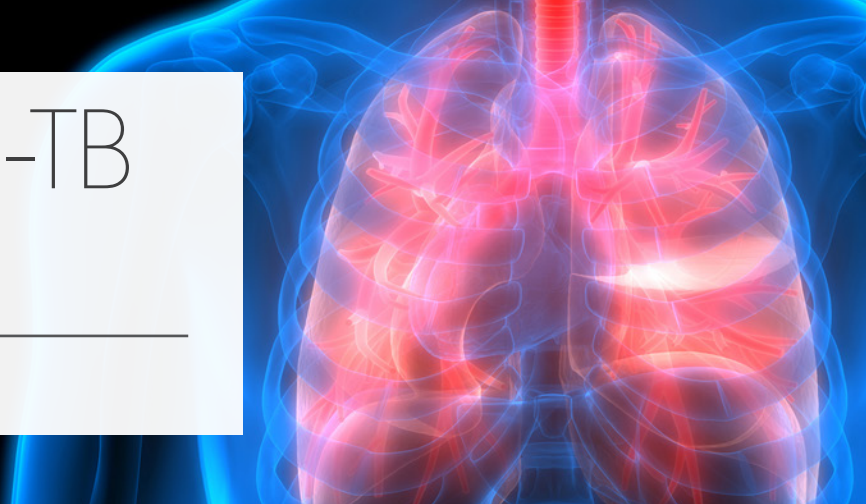

QuantiFERON-TB Gold Plus, 1-Tube (3017554)


Collection instructions:
Collect 6 mL (5 mL min)
of whole blood drawn into
an ARUP-provided single
lithium-heparin tube
(ARUP supply #54015).

Specimen must remain
ambient for a minimum of
15 minutes after collection
before being refrigerated.
Ship refrigerated.



**Transport to ARUP same day
as collected. Specimen must
reach ARUP's testing lab
within 48 hours of collection.**

**ARUP performs all
of the specimen
preparation steps.**

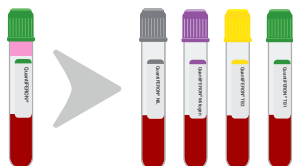


ARUP transfers 1 mL of
whole blood from single
lithium-heparin tube into four
QFT-Plus tubes.

QuantiFERON-TB Gold Plus, 4-Tube (3017562)

Collection instructions:
Collect 4 mL of whole
blood—1 mL drawn into
each of the **four** QFT-Plus
blood collection tubes (ARUP
Standard supply #54012 or
High Altitude supply #54010).

Process (incubate and
centrifuge) in-house.
**Ship refrigerated to
ARUP after processing.**

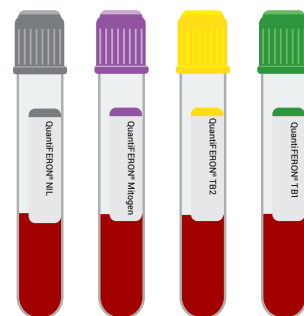

**Client must perform
all of the specimen
preparation steps.**

Transport to ARUP.



**Testing performed at
ARUP and results reported.**

View our testing options:

aruplab.com/testing/quantiferon

© 2024 ARUP LABORATORIES | BD-TMC-029, REV 1 | MAY 2024 PAGE 2 OF 2