



## STOOL PRESERVATION COLLECTION AND TRANSPORTATION CHART

Unless otherwise indicated, if a test has more than one preferred or acceptable sample type marked, only ONE sample needs to be sent.

ARUP Supply Number:



40910

Preservative:

Unpreserved



52059

Parasep



29799

Enteric Transport Media (Cary-Blair)







49952

FOBT-Sample Bottle

ARUP Test Code/Mnemonic	Test Name	Temperature:	Refrigerated		Frozen		Room Temperature	Refrigerated	
			Preferred	Accepted	Preferred	Accepted	Preferred	Accepted	
<b>ANTIGEN TESTING</b>									
2011041/A1A RANDOM	Alpha-1-Antitrypsin, Quantitative by ELISA, Random Stool		Preferred		Preferred				
2011043/A1ACLR	Alpha-1-Antitrypsin Clearance		For detailed collection and transport instructions, visit <a href="http://aruplab.com/Specimen-Handling/resources/CollectionShipping/StoolSample_Instructions">aruplab.com/Specimen-Handling/resources/CollectionShipping/StoolSample_Instructions</a> .						
3002859/CALPRO FEC	Calprotectin, Fecal by Immunoassay		Preferred		Preferred				
0058002/CAMPY	Campylobacter Antigen		Preferred		Preferred		Preferred		
0060045/CRYSPO	Cryptosporidium Antigen by EIA				Preferred				
0060047/ECSTL AG	<i>E. coli</i> Shiga-like Toxin by EIA						Preferred		
0058001/AMOEB	<i>Entamoeba histolytica</i> Antigen, EIA				Preferred				
0060048/GLAM	<i>Giardia</i> Antigen by EIA				Preferred		Accepted, but not preferred		
0065147/PYLORI AG	<i>Helicobacter pylori</i> Antigen, Fecal by EIA		Accepted, but not preferred		Preferred				
0061164/FECLACTO	Lactoferrin, Fecal by ELISA		Preferred		Preferred		Preferred		
2007190/FOB IA	Occult Blood, Fecal by Immunoassay								Preferred
3002858/ELASTASE	Pancreatic Elastase, Fecal by Immunoassay		Preferred		Preferred				
0065088/ROTA	Rotavirus Antigen by EIA		Accepted, but not preferred		Preferred				
<b>CULTURES</b>									
0060851/CDIFTX	<i>Clostridium difficile</i> Cytotoxin Cell Assay		Preferred		Preferred				
0060140/MC CDIF	<i>Clostridium difficile</i> Culture with Reflex to Cytotoxin Cell Assay				Preferred		Accepted, but not preferred		
0060135/MC CAMP	Stool Culture, <i>Campylobacter</i>						Preferred		
0060134/MC SSC	Stool Culture and <i>E. coli</i> Shiga-like Toxin						Preferred		
0060136/MC VIB	Stool Culture, <i>Vibrio</i>						Preferred		
0060137/MC YERS	Stool Culture, <i>Yersinia</i>						Preferred		
<b>ELECTROLYTE/OSMOLALITY TESTING</b>									
0060185/FEC LYLES	Electrolytes, Fecal		Preferred		Preferred				
0020699/FEC PRO	Electrolyte and Osmolality Profile, Fecal				Preferred				
0020381/FE-CL	Chloride, Fecal		Preferred		Preferred				

## FECAL PRESERVATION COLLECTION AND TRANSPORT CHART

Unless otherwise indicated, if a test has more than one preferred or acceptable sample type marked, only ONE sample needs to be sent.

		ARUP Supply Number:					
		Preservative:		40910	52059	29799	49952
				Unpreserved	Parasep	Enteric Transport Media (Cary-Blair)	FOBT-Sample Bottle
ARUP Test Code/Mnemonic	Test Name	Temperature:	Refrigerated	Frozen	Room Temperature	Refrigerated	
0020105/FECMG	Magnesium, Fecal		Preferred	Preferred			
0020380/FE-K	Potassium, Fecal		Preferred	Preferred			
0020379/FE-NA	Sodium, Fecal		Preferred	Preferred			
0098122/FEC OSMO	Osmolality, Fecal			Preferred			
<b>FECAL FAT TESTING</b>							
0020385/FECFAT	Fat, Fecal Qualitative		Preferred	Preferred			
2002354/FECQFAT24	Fat, Fecal, Quantitative, 24-Hour Collection (Includes Homogenization)		For detailed collection and transport instructions, visit <a href="http://aruplab.com/Specimen-Handling/resources/CollectionShipping/StoolSample_Instructions">aruplab.com/Specimen-Handling/resources/CollectionShipping/StoolSample_Instructions</a> .				
2002355/FECQFAT48	Fat, Fecal, Quantitative, 48-Hour Collection (Includes Homogenization)						
2002356/FECQFAT72	Fat, Fecal, Quantitative, 72-Hour Collection (Includes Homogenization)						
2002350/FECQ FAT	Fat, Fecal Quantitative, Homogenized Aliquot		Accepted, but not preferred	Preferred			
<b>PCR</b>							
2002838/CDIFF PCR	<i>Clostridium difficile</i> Toxin B Gene ( <i>tcdB</i> ) by PCR		Preferred	Preferred			
3001801/CDIFF LFA	Toxigenic <i>Clostridioides difficile</i> by LFA with Reflex to PCR, Stool		Preferred	Preferred			
3003279/GIPPCR	Gastrointestinal Pathogens Panel by PCR					Preferred	
2011150/GI PARAPCR	Gastrointestinal Parasite Panel by PCR		Accepted, but not preferred	Preferred			
2011626/MICROSPCR	Microsporidia by PCR		Accepted, but not preferred	Preferred			
2014546/NOROPCR	Norovirus, Groups 1 and 2 by PCR		Accepted but not preferred	Preferred			
<b>STAIN/MICROSCOPY/MANUAL TESTING</b>							
0060050/MICROST	Microsporidia Stain by Modified Trichrome				Preferred		
3001662/OP FEC	Ova and Parasite Exam, Fecal (Immunocompromised or Travel History)				Preferred		
0060046/PARAST	<i>Cryptosporidium</i> and <i>Coccidia</i> Exam, Fecal				Preferred		
3002514/FEC RED	Reducing Substances - Fecal		Accepted, but not preferred	Preferred			
0020518/FEC-PH	pH, Fecal		Accepted, but not preferred	Preferred			